

Instructions for use


Product description:

Alcohol Free Surface Disinfectant


Date of revision: 16/11/23 v1.6

topdental™
Bossklein®


Specially formulated for use in the medical and dental environment. Contains detergent. For the surface disinfection of medical devices, including but not limited to, dental chairs, examination lights and patient transport. Not intended for use on medical and dental instruments.

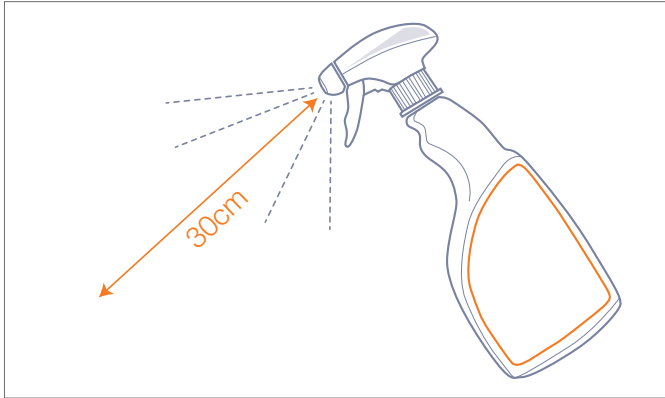

1. Spray clean surfaces and objects to be disinfected from a distance of 30cm so that they are completely moistened.

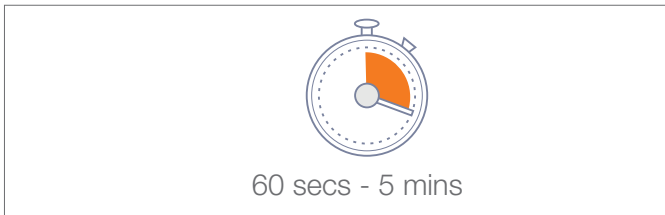

2. Let the disinfectant take action over 60 seconds (to achieve full efficacy leave for 5 minutes).

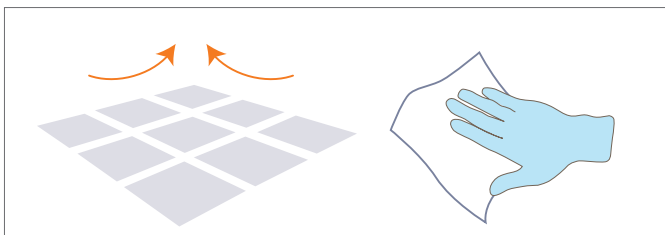

3. Allow surface spray to dry or wipe with clean tissue. For professional use only. See product labels and MSDS for further information.

Efficacy: Bactericidal (5mins). Yeastocidal (5 mins). Virucidal activity against enveloped viruses (60 secs): Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to standards: EN 16777, EN 16615, EN 14885, EN 14476, EN 13727, EN 13624.

Composition: Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecyltrimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

Warning. H412: Harmful to aquatic life with long lasting effects. First aid procedures. P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Handling and Storage. Ensure there is sufficient ventilation of the area. The wearing of protective gloves is recommended. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool.

Contraindications. Incompatible materials: mild steel, aluminium, copper.

Disposal. Dispose of contents/container in accordance with local regulations. P273: Avoid release to the environment. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

Intended Users. The products are used on surfaces within the dental surgery by the dentist or dental assistant as part of their general disinfecting routine in-between patient appointments. Patients are usually not present during use, but all patients that are seen in the surgery contribute to the need for the products.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

UFI: 2520-0057-K00H-5Q24

